



Start injecting with our strongest direct restorative ever

G-ænial® Universal Injectable

High-strength restorative composite

- Injectable with total control
- Superb handling with excellent adaptation
- Strong and highly durable restorations for any cavity class
- Long-lasting aesthetics

Welcome to the future



Innovative syringe design delivers with ease and no slumping

No stringing, no slumping, no stress



Bendable tips for excellent cavity adaptation



Easy extrusion for maximum comfort

3 Unique Characteristics

Unsurpassed aesthetics

Universal indications

Direct injection delivery

Unsurpassed Polishability with Beautiful Aesthetics

Universal Application for all Cavity Classes

Faster Placement Time with No Waste



Class I



Class II



Class III



Class IV



Class V

G-aenial Universal Injectable

3 in 1 Technologies

Ultra-fine
Fillers

Full-Coverage
Silane Coating

Improved
Resin
Monomer

This technology gives unparalleled strength, durability, energy absorbing resilience with high radiopacity to the matrix.

3 in 1 Benefits

Exceptional
Strength

Unsurpassed
Surface
Integrity

Long
Lasting
Aesthetics

3 in 1 Savings

Less Time

No Waste

Less Stress

Injection Moulding Technique



1. Initial situation



2. Fill the impression tray with EXACLEAR and take an impression of the wax-up on the model



3. After setting, remove the clear silicone from the tray



4. Drill a hole through the silicone key to create space for the composite tip



5. Isolate the neighbouring teeth, etch the enamel and apply a bonding agent



6. Place the silicone key in the mouth and inject G-aenial Universal Injectable through the hole in the key



7. Light-cure through the silicone key and then remove it



8. Remove excesses of composite with a blade



9. Final result after a few polishing steps

Conventional pastes



G-aenial™ Universal Injectable



Posterior Restorations



Dr Javier Tapia Guadix - Spain

Packaging



Contains:
1 x 1.7g (1.0mL) Syringe

Shades:
A2 & A3