GC Fuji ORTHO BAND PASTE PAK

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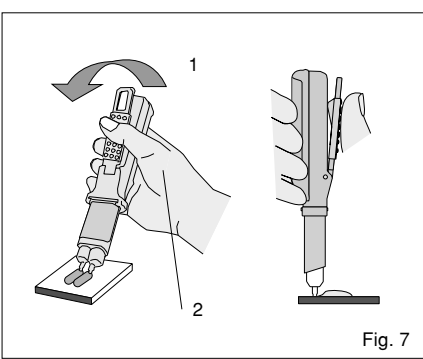
Fig. 7

GC Fuji ORTHO BAND PASTE PAK

GLASS IONOMER ORTHODONTIC BAND CEMENT

GC Fuji ORTHO BAND PASTE PAK is the first glass ionomer orthodontic band cement available in paste-paste presentation. GC Fuji ORTHO BAND PASTE PAK is available in a cartridge pack and can be dispensed easily and correctly using a special dispenser called Paste Pak Dispenser.

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

1. Cementation of orthodontic bands.
2. Cementation of appliances retained with stainless steel crowns (e.g., Herbst or expansion appliances).

CONTRAINDICATIONS

In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product and refer to a physician.

IDENTIFICATION OF PARTS

- (Paste Pak Cartridge)
1. Cover
 2. Inner plunger
 3. Tip
- (Paste Pak Dispenser)
4. Piston
 5. Finger grip Long
 6. Lever
 7. Slide block
 8. Release lever (Fig. 1)

DIRECTIONS FOR USE

Paste / Paste Ratio (g/g)	2.0 / 1.0
Mixing time (sec.)	15-20*
Working time (min. sec.)	3'30*
Net setting time (min. sec.)	3'00*

Test conditions : Temperature (23 +/- 1°C) Relative humidity (50 +/- 5%)
*Test Method based on ISO 9917:1991 (E) (Glass polyalkenoate cement)

1. Paste Pak Cartridge Loading

- a. Make sure that the piston is completely retracted into the dispenser. (Fig. 2)

Note :
To retract the piston, hold both the slide block and release lever (on the underside of the dispenser) with fingers and pull them back together.

- b. Load the cartridge into the Paste Pak Dispenser. (Fig. 3)
1) Ensure the ▲ mark on the cartridge lines up with the ▲ mark on the Paste Pak Dispenser as illustrated.
2) Then turn the cartridge fully to the right position.

- Note :
To remove the cartridge, retract the piston fully and turn the cartridge back to the marked positions.
- c. After cartridge loading, slide the release lever forward until it stops. (Fig. 4)
1) Push the front of the release lever.
2) Do not push the upper side.

2. Dispensing

- a. Remove the cartridge cover. (Fig. 5)
b. Depress the lever to dispense the required amounts of pastes onto the mixing pad.

Note :
1) Before dispensing the first material from a new cartridge or when dispensing after a long interval, bleed the pastes in order to prevent the incorporation of air bubbles into the material.

- 2) The finger grip can be moved forwards or backwards to change the quantity of material to be dispensed. (Fig. 6)
1. Position of Finger grip Long for a smaller quantity.
2. Position of Finger grip Long for a larger quantity.

- c. While depressing the lever, level and cut off the extruded material from the tips of the cartridge by moving the cartridge and dispenser to an upright position on the mixing pad. (Fig. 7)
1. Level and cut off material.
2. Depressing the lever.

When the lever is released after dispensing, residual pastes at the tips are retracted into the cartridge. Replace the cartridge cover.

- 1) If any surplus paste is retained on the cartridge tips after dispensing, remove with gauze, etc.
- 2) The amount of residual material in the cartridge can be confirmed by the position of the slide block on the underside of the dispenser. As the cartridge has translucent tips, the residual amount can also be confirmed by the position of the inner plunger seen through the tips.
- 3) Do not drop the dispenser with the cartridge. Otherwise the cartridge may be damaged.

3. Mixing

After dispensing, incorporate and spread the material out in a thin layer on the mixing pad using a plastic spatula. Mix thoroughly, with lapping strokes, for 15-20 seconds. Be careful not to incorporate air bubbles.

- Note :
1) The working time is 3 minutes 30 seconds from the start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time. Beware that the cement will set sharp in the mouth.
2) Do not mix pastes with other materials.

4. Band preparation

- a. After adapting the bands, remove any contaminants from the internal surfaces.
- b. For improved retention, roughen the internal surfaces of the bands with a rotary instrument or by sandblasting or microetching.

5. Tooth cleaning

- a. Clean the tooth surfaces to be cemented with pumice and water.

- b. Rinse thoroughly. Blot away excess water with a cotton pellet or sponge. **DO NOT DESICCATE.** Enamel surfaces should appear moist (glistening).

6. Cementation technique

- a. Bands
1) Using a suitable instrument, place the material throughout the internal surfaces of the bands.
2) Seat the bands.
3) When bands are in the final position, begin removing excess cement after initial set.

- b. Appliances retained with stainless steel crowns
1) To facilitate removal of the appliance, coat the occlusal surfaces of the teeth with a fluoride gel or petroleum jelly prior to cementation.
2) Fill the adapted crown with cement and seat. Using a brush or sponge, begin removing excess cement after initial set.

7. Debonding procedure

Remove bonded bands from the buccal or lingual side using band removing pliers. Then remove residual cement on the tooth using adhesive removing pliers. And further, remove a thin layer of cement adhering to the tooth using an ultrasonic scaler.

Cleaning / Sterilization of Paste Pak Dispenser

1. Clean the dispenser and the finger grip with gauze soaked in alcohol.
2. The dispenser may be steam autoclaved at 121-135°C (250-275°F) for 10-30 minutes or disinfected using alcohol. (Please remove the finger grip and the cartridge before autoclaving the dispenser.) The use of various chemical disinfectants / sterilants may cause damage to the dispenser and therefore is not recommended.

Note :
After autoclaving, the release lever of the dispenser may not easily be moved forwards and backwards by fingers. In such a case, slide the release lever forward until it stops. Apply a very small quantity of petroleum jelly onto the entire surface of the piston. Do not apply excess petroleum jelly. Then move the release lever back and forth a few times to ensure smooth movement.

SHADE

Blue

STORAGE

Store in a cool and dark place (4 - 25°C) (39.2 - 77.0°F) (Shelf life : 2 years from date of manufacture)

PACKAGE

Paste Pak Cartridge (13.3g / 7.2mL) (2) with mixing pad (No.22)
Option : Paste Pak Dispenser (1)
*Starter Kit (US only)
Paste Pak Cartridge (13.3g / 7.2mL) (2), Paste Pak Dispenser (1), mixing pad(No.22) and plastic spatula

CAUTION

1. Avoid excessive contact with oral tissues, removing excess with cotton/gauze and rinsing with water after initial set.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. In rare cases metal may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of Paste Pak Dispenser (nickel-plated aluminum) and refer to a physician.
4. Store Paste Pak Dispenser away from chemicals such as iodine tincture containing halogen element. Otherwise the dispenser may discolor.

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