Prior to use, carefully read the instructions for use.

EN

GC Fuji PLUS CONDITIONER

TOOTH CONDITIONING AGENT

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

GC Fuji PLUS CONDITIONER is a tooth conditioning agent designed for use with GC Fuji PLUS glass ionomer luting cement. It helps to enhance the bond between GC Fuji PLUS and tooth structure.

CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

1. After tooth preparation, apply GC Fuji PLUS CONDITIONER to the bonding surface for 20 seconds using a cotton pellet or sponge.
2. Rinse thoroughly with water and dry. DO NOT DESICCATE.
3. Proceed according to GC Fuji PLUS directions.
4. Close the bottle immediately after use.

STORAGE

Store at temperature of 4-25°C (39.2-77.0°F). (Shelf life: 2 years from date of manufacture)

PACKAGES

Bottle of 7g (6.5mL)

CAUTION

1. In case of contact with oral tissue or skin, flush immediately with water.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. GC Fuji PLUS CONDITIONER contains 2% Ferric Chloride. Not recommended for use with products that contain Epinephrine, which is known to react with iron containing products and may stain oral tissue.

Last revised : 05/2008

 0086

MANUFACTURED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

 EU: GC EUROPE N.V.
Research-Park, Interleuvenlaan 13, B-3001 Leuven, Belgium
TEL: +32. 16. 39. 80. 50.

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900

GC ASIA DENTAL PTE. LTD.
19 Loyang Way, #06-27 Singapore 508724
TEL: +65 6545 7588

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