MI Paste™

Topical paste with bio-available calcium and phosphate

For use only by a dental professional in the recommended indications.

DESCRIPTION

MI Paste™ is a water based topical paste containing Recaldent™™ (CPP-ACP : Casein Fosfopeptíde – Amorphous Calcium Phosphate). It is a unique blend of polishing, cleaning and dental tubule sealing agents designed for professional application during standard dental practice hygiene procedures. When CPP-ACP is applied in the oral environment, it will bind to biofilms, plaque, bacteria, hydroxyapatite and soft tissue, localizing bio-available calcium and phosphate.

IMPORTANT

Saliva will enhance the effectiveness of CPP-ACP and the flavor will help stimulate saliva flow. The longer CPP-ACP and saliva are maintained in the mouth, the more effective the result.

RECOMMENDED INDICATIONS

To be used for cleaning and polishing procedures as part of a professionally administered prophylaxis treatment. Secondly, MI Paste™ can be used for the management of tooth sensitivity, post-scaling, root planing and bleaching.

CONTRAINDICATIONS

Recaldent™™ (CPP-ACP) is derived from milk casein.

Do not use on patients with a milk protein and/or hydroxybenzoates allergy.

DIRECTIONS FOR USE

I. Prophy Cup Application

- Following routine prophylaxis for removal of plaque, food debris and stain, ask the patient to rinse.
- Apply a generous layer of MI Paste™ as a final finishing paste to the tooth surfaces using a prophy cup, prophy brush, a gloved finger or in difficult interproximal areas using an interproximal brush.
- Request the patient to hold in the mouth for as long as possible (1-2 minutes) avoiding expectoration and delaying swallowing. The longer MI Paste™ and saliva are maintained in the mouth, the more effective the result.
- Advise the patient not to eat or drink for 30 minutes following application.

Note :

Apply the paste to the teeth - especially at the cervical surfaces where most tooth sensitivity occurs.

II. Custom tray application

- Management of tooth sensitivity
- Rinse the custom tray thoroughly under running water.
- Extrude a generous layer of MI Paste™ into the tray and apply to the upper and / or lower teeth.
- Leave the tray undisturbed in the mouth for a minimum of 3 minutes.
- Remove the tray.
- Instruct the patient to use the tongue to spread the remaining MI Paste™ throughout the mouth. Instruct the patient to retain for as long as possible (1- 2 minutes) avoiding expectoration and delaying swallowing. The longer MI Paste™ and saliva are maintained in the mouth, the more effective the result.
- Ask the patient to rinse any MI Paste™ remaining on the surface. Any residual MI Paste™ can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.
- Any residual MI Paste™ in the tray should be rinsed or brushed off under running water immediately after use.

Note :

Non tray application – Apply a generous layer of MI Paste™ to the tooth surfaces as required. Follow Steps 5 and 6 as above.

FLAVORS

Strawberry (S), Melon (M), Vanilla (V), Mint (I), Tutti-Frutti (T)

Note: S, M, V, I and T in parentheses are abbreviations for each flavor. These abbreviations are marked on the tubes after lot numbers to show the flavor of the contents. For example, 021003S.....Strawberry

STORAGE

Store in a cool dry low humidity area away from direct sunlight (8 - 25°C) (46.4 – 77.0°F). (Shelf life: 2 years from date of manufacture)

PACKAGE

Tube containing 40g (35mL) Assortment Package: 10 x 40g tubes (2 each of Strawberry, Melon, Vanilla, Mint, Tutti-Frutti)

Vanilla is especially recommended for patients suffering from erosion caries and conditions arising from xerostomia.

NOTE

MI Paste™ with its unique Recaldent™ (CPP-ACP) formulation is available through the dental profession and is not sold in pharmacies, supermarkets or other retail outlets.

*CPP-ACP was developed at The School of Dental Science, The University of Melbourne Victoria Australia.

Recaldent™ is a registered trademark of Recaldent Pty Ltd. CPP-ACP technology has related patents or patents pending in Australia, NZ, Europe, Canada and USA.

CAUTION

- Always replace the cap after use and remove any residual paste or moisture around the tube neck or inside the cap.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- In case of spillage on clothes, wash off with water.
- If any angioedema symptoms are experienced, discontinue the use of the product and refer to a physician.

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