



Making a great
IMPRESSION

with


IMPRECEED

Dust-free Impression Material

GC

IMPREECED

From powder to perfect paste in just 20 seconds

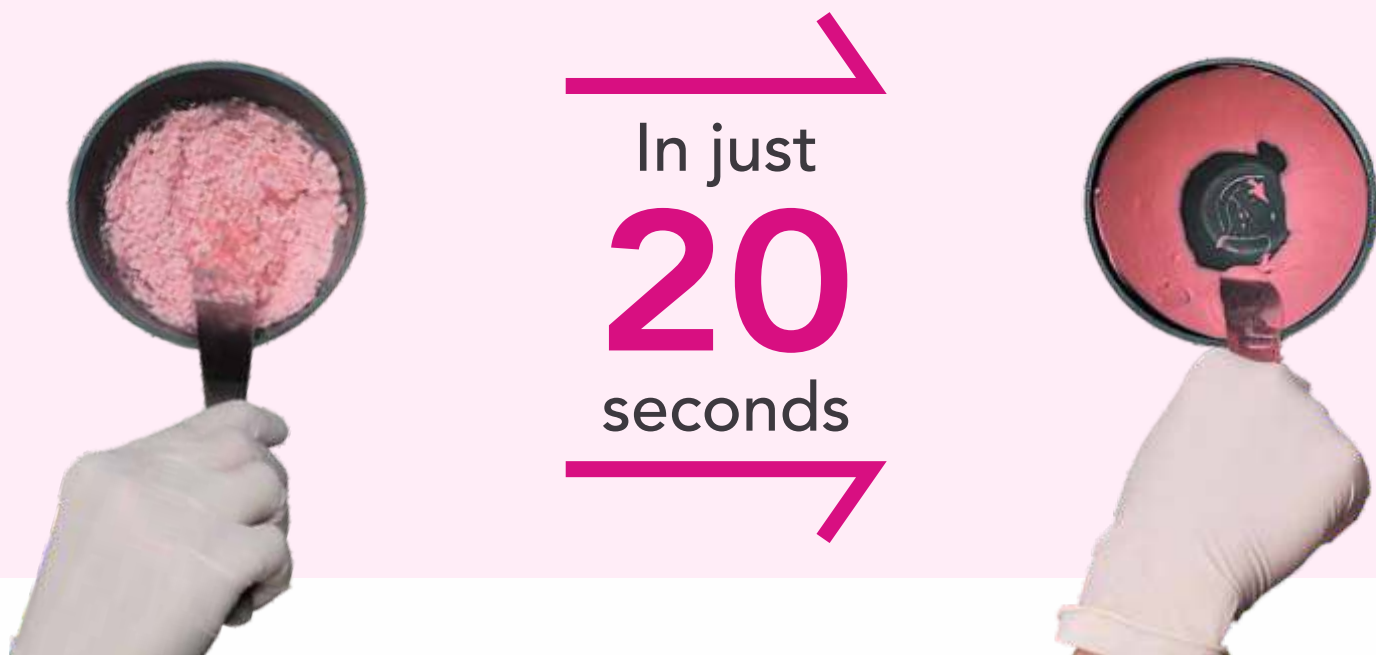

The new IMPREECED powder is incorporated rapidly into water through enhanced powder-water affinity, allowing more spatulation time to obtain a smooth bubble-free mix within 20 seconds.

IMPREECED



The reliable properties of IMPREECED remain unchanged.

Accurate impression



Easy-mixing of IMPREECED results in a creamy, bubble-free consistency that captures details accurately.

Optimal thixotropic property



IMPREECED provides a balance of good drip resistance and fluidity to effectively provide coverage enabling detailed impression taking with minimal discomfort to patient.

Smooth model surface



IMPREECED has good compatibility with hard stones producing smooth and precise detail reproduction on the gypsum model.

IMPRECEED Physical Properties

Setting types	Compressive strength	Strain in compression	Recovery from deformation
Normal	0.7MPa	14.6%	97.4%

Mixing time	Minimum time in mouth	Setting time
20-30 sec.	120 sec.	180 sec.

Recommended Indications

Preliminary impressions.

Impressions for the fabrication of:

- models for temporary inlays, onlays, crowns and bridges
- orthodontic models
- study models

Directions for use

1.

8.4g + 20mL =

16.8g + 40mL =

Measurement guidelines

2.

20-30 sec

Spatulate vigorously

3.

Normal
Approx 120 sec

Hold tray in the mouth. Refreshing spearmint smell for patient comfort

4.

Rise the impression under running water, and remove excess water

5.

Pour Alginate Impression as soon as possible

6.

If cast cannot be poured immediately, wrap the impression in a wet paper towel and store in a tightly sealed plastic bag.

Note: Recommended for optimal performance, store in a cool dry place, in a tightly closed container. Loosen the powder well before every use.

The setting time includes the mixing time and the minimum manipulation time.

This is based on the mixing at 23°C with distilled or deionized water.

Higher temperatures of the mixing water will shorten the setting time and total working time.

Variations in hardness of the water may influence the setting time.

Packaging

Impreceed - 454 g



Accessories

COE Spacer Trays



COE Spacer Trays



- 1D Large Upper (Bag of 12)
- 4D Medium Upper (Bag of 12)
- 7D Small Upper (Bag of 12)
- 20D Large Lower (Bag of 12)
- 21D Medium Lower (Bag of 12)
- 22D Small Lower (Bag of 12)
- Partial 30D U/Lft, L/Rt, (Bag of 12)
- Partial 31D U/Rt, L/Lft, (Bag of 12)
- Partial 32D Anterior, (Bag of 12)



GC INDIA DENTAL PVT. LTD.

#233, Phase 3, IDA Pashamylaram,
Sangareddy District - 502307
Telangana, India. Tel.: +91 8455 224 844
Email: info.gcindia@gc.dental
Web: www.gcindiadental.com

Connect us on:

-  <https://www.facebook.com/GcIndiaDental1>
-  https://twitter.com/GCIndia_Dental
-  <https://www.linkedin.com/in/gcindiadental/>
-  <https://www.instagram.com/gcindiadental/>
-  <https://www.youtube.com/channel/UCyvjHQB0aSk2tHEgC-KLw>

