Prior to use, carefully read the instructions for use.

EN

GC Tri Plaque ID Gel™

DENTAL PLAQUE DISCLOSING GEL

For use only by a dental professional in the recommended indications.

DESCRIPTION

GC Tri Plaque ID Gel will differentiate among fresh plaque, mature plaque (more than 48 hours old) and strong acid producing plaque by colour to aid in patient oral hygiene education.

CONTRAINDICATIONS

Do not use this material on patients with a sensitivity or allergy to benzoate preservatives.

DIRECTIONS FOR USE

1. Apply GC Tri Plaque ID Gel onto the tooth surfaces using a suitable instrument (microbrush, swab or tooth brush). Dispense gel into a dish if you apply on multiple tooth surfaces. Following this, instruct the patient to lightly rinse their mouth with water. Identify the plaque by observing the color of the gel.
 - a) A pink or red colour on the tooth surface indicates fresh plaque accumulation.
 - b) A blue or purple colour indicates mature plaque that is at least 48 hours old.
 - c) A light blue color indicates mature and strong acid producing plaque.

Note:
Clean the tip of the tube by wiping with a sponge or cotton pellet to remove any excess gel. Close the cap immediately.
2. Instruct the patient in proper tooth cleaning and thereby remove any remaining plaque and disclosing gel.

STORAGE

Store in a cool dry low humidity area away from direct sunlight (8-25°C). Shelf life: 3 years from date of manufacture.

PACKAGE

Tube containing 40g (36mL)

CAUTION

1. Avoid contact with eyes. In case of contact, immediately flush with water and seek medical attention.
2. Avoid contact with skin, clothing or equipment. In case of contact, immediately wash with water.
3. Avoid contact with dental cavity. If case of contact, immediately wash with water. And if the area are stained, the color will be gradually removed.
4. This product contains red and blue dyes, which will stain dental restorative materials.
5. Do not swallow.
6. Do not mix with other products.
7. Keep out of reach of children.
8. If any angioedema symptoms are experienced, this may indicate sensitivity or allergy to the benzoate preservatives. In this event, discontinue the use of the product and refer to a physician.

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