## GC Fuji IX GP EXTRA

RADIOOPAQUE POSTERIOR GLASS IONOMER RESTORATIVE CEMENT IN CAPSULES

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

- Class I and II restorations in deciduous teeth.
- Non-load bearing Class I and Class II restorations in permanent teeth.
- Intermediate restorative and base material for heavy stress situation in Class I and Class II cavities using sandwich laminate technique.
- Class V and root surface restorations.
- Core build-up.

### CONTRAINDICATIONS

- Pulp capping.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.


### DIRECTIONS FOR USE

Powder / Liquid Ratio (g/g)	0.40 / 0.12
Mixing Time (sec.)*	10"
Time of initial extrusion after mixing	10 seconds maximum
Working Time (min., sec.)*	1'15"
Net Setting Time (min., sec.)*	2'00"
Final Finishing Commencing Time (37°C, after starting mix) (min.,sec.)	2'30"

Test conditions : Temperature (23 +/-1°C), Relative humidity (50 +/-5%)  
\*ISO 9917-1 : 2003 (E) (Glass polyalkenoate cement)





### 1. CAVITY PREPARATION

- Prepare tooth using standard techniques. Extensive mechanical retention is unnecessary. For pulp capping use calcium hydroxide.
- Apply GC CAVITY CONDITIONER (10 seconds) or GC DENTIN CONDITIONER (20 seconds) to the bonding surfaces using a cotton pellet or sponge (Fig. 1).
- Rinse thoroughly with water. Blot away excess water with a cotton pellet or dry by gently blowing with an air syringe. DO NOT DESICCATE. Prepared surfaces should appear moist (glistening).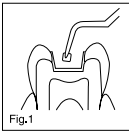




### 2. CAPSULE ACTIVATION AND MIXING

- Before activation, shake the capsule or tap its side on a hard surface to loosen the powder (Fig. 2).
  - To activate the capsule, push the plunger until it is flush with the main body (Fig. 3).
  - Immediately place the capsule into a metal GC Capsule Applier and click the lever once (Fig. 4). The capsule is now activated.
- Note :
- The capsule should be activated just before mixing and used immediately.
- Immediately remove the capsule and set it into a mixer (or an amalgamator) and mix for 10 seconds (+/- 4,000RPM) (Fig. 5).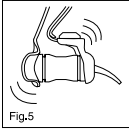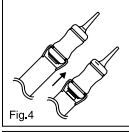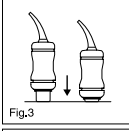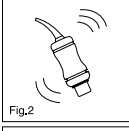

### 3. RESTORATIVE TECHNIQUE

- Immediately remove the mixed capsule from the mixer and load it into the GC Capsule Applier.
  - Make two clicks to prime the capsule then syringe (Fig. 6). The working time is 1 minute 15 seconds from start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time.
  - Within 10 seconds maximum after mixing, start to extrude the mixture directly into the preparation (Fig. 7).
  - Form the preliminary contour, and cover with a matrix if required.
  - When set, immediately apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure) (Fig. 8).
- Note :
- TO ADJUST THE DIRECTION of the nozzle, hold the applier with the capsule towards you and turn the capsule body.
  - To remove the used capsule, push the applier release button. Twist the capsule and pull upwards.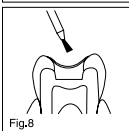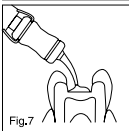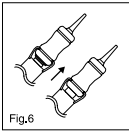

### 4. FINISHING

- Final finishing under water spray using standard techniques can begin approximately 2 minutes 30 seconds after start of mixing (Fig. 9).
- Apply a final coat of GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure for 10 sec.) or G-COAT PLUS (light cure for 20 sec.) (Fig. 10).
- Instruct the patient not to apply pressure for 1 hour.

### STORAGE

Store the capsules in the original aluminium foil in a cool and dark place (4-25°C) (39.2-77.0°F).  
(Shelf life : 2 years from date of manufacture).

### SHADES

A1, A2, A3, A3.5, B1, B2, B3, C4  
Shade numbers according to Vita® shade guide.  
Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.


### PACKAGES

- Single shade package : 50 capsules.
- Assortment package : 50 capsules (10 each of A2, A3, A3.5, B1, B3).  
Average contents per capsule : 0.40g powder and 0.12g (0.10mL) liquid.  
Minimum net volume of mixed cement per capsule : 0.14mL.

Option : GC CAPSULE APPLIER (1 piece).

### CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.

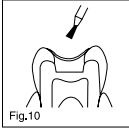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