GC PLAQUE INDICATOR KIT

FOR IDENTIFICATION OF CARIOGENIC FERMENTATION PLAQUE

For use only by a dental professional in a recommended indications.

GC PLAQUE INDICATOR KIT is an in vitro patient motivation test kit, containing a Plaque Indicator Solution to determine the pH of plaque when subjected to a sucrose challenge. The Plaque Disclosing Gel will differentiate between fresh plaque adhering to the tooth surface and mature plaque (more than 48 hours old) to aid in patient oral hygiene education. The kit also contains a Plaque Indicator Neutralizing Solution for encouraging patient motivation to improve his/her current oral condition. PLAQUE INDICATOR KIT will assist the dental team in communicating with patients by visually demonstrating the cariogenic potential (fermentation ability) of dental plaque and tooth surfaces that need more careful attention during oral hygiene to help avoid future caries. The test assesses total acid production and is not only restricted to lactic acid. It is responsive to the protective buffering effects of weak organic acids, and thus gives a global assessment of how readily the plaque can produce acids when exposed to sucrose.

RECOMMENDED INDICATIONS

- To educate patients on plaque cariogenicity
 - * as a baseline for new patients needing restorative care
 - * for patients suspected of having a high caries risk
 - * during orthodontic treatment
 - * during complex or extended dental treatments.
- To demonstrate plaque acid production to a patient as part of their education about caries.
- To evidence the effect of dietary changes (such as a reduced intake of acidic drinks) on plaque pathogenicity.

- The test can be repeated after 1 or 2 weeks to see the impact of dietary changes.
- It should be repeated at least every 2-3 months in orthodontic patients with visible plaque because of the risk of decalcification in such patients.
- This test can be useful in all patients regardless of age.

DIRECTIONS FOR USE

Precaution:

- Do not use Plaque Disclosing Gel or other disclosing agents before collecting any plaque samples as this may cause an inaccurate result. Plaque contaminated by blood is not suitable for the test and may cause an inaccurate result.
- Use the gel and solutions immediately after dispensing.

DISPENSING COMPONENTS

- Dispense two drops of Plaque Indicator Solution (A) into the disposable dispensing dish provided (Section A, Fig. 1).
Note: Use immediately after dispensing.
- Dispense two drops of Plaque Indicator Neutralizing Solution (B) into the disposable dispensing dish (Section B, Fig. 1).
- Dispense one drop of the Plaque Disclosing Gel (C) into the dispensing dish (Section C, Fig. 1).
- Prior to use, disinfect the disposable collection instrument (s) by brief contact (5 seconds) with alcohol.

PLAQUE COLLECTION

- Using the disposable plaque collection instrument(s), harvest plaque from interproximal ites in the molar area (preferably in the mandible).
If visible plaque is also present on the buccal surfaces of the mandibular canines and premolars, or on the labial aspect of the maxillary incisors, this should also be sampled, using a separate collection instrument. It is only necessary to collect plaque from one side of the mouth as the fermentation ability does not vary substantially from one side to the other.
- To achieve an accurate result, collect sufficient plaque to fill at least half of the tip of each instrument used (Fig. 2).
Note:
Each PLAQUE INDICATOR KIT allows for two instruments per patient to be used. Immediately prior to collecting the plaque sample, use an air syringe to lightly dry the site to be sampled, to reduce the risk of contamination with saliva (which may cause an inaccurate result).
- Dip the instrument(s) with attached plaque into the Plaque Indicator Solution for 1 second ONLY. The surface of the plaque will turn green.
Note:
Plaque samples that have been immersed in the solution for too long will provide an inaccurate result.
- Place the instrument(s) into the groove(s) of the dispensing dish and leave the samples to ferment for 5 minutes (Fig. 3).
- At 5 minutes, measure the pH value by checking the colour and compare with the chart on the dispensing dish.
Note:
The plaque sample(s) will change colour over time. As the greatest acid production occurs at 5 minutes, this is the best time to check the final pH level.

APPLICATION OF THE PLAQUE DISCLOSING GEL

Whilst waiting for the plaque sample to ferment, apply the Plaque Disclosing Gel to the tooth surfaces using a microbrush. Following this, instruct the patient to lightly rinse their mouth with water.
Observe any disclosing gel coating on the tooth surface.

- A pink or red gel colour on the tooth surface indicates fresh plaque accumulation.
- A blue or purple gel colour indicates mature plaque that is at least 48 hours old.

CHECKING AND INTERPRETING THE RESULTS

- At 5 minutes, check the fermented plaque sample(s).
- a) A green colour indicates a neutral pH around 7.2.
The plaque has a low fermentation ability and the pH has been unaffected by the sucrose challenge.
- b) A yellow or orange colour indicates a final pH of 6.0-6.6.
- c) A pink or red colour indicates a final pH of 5.0-5.8.

For yellow or red coloured plaque samples, preventive action is recommended. This should include discussion with the patient about the dangers of plaque acid production which can lead to prolonged demineralization and possible cavitation from dental caries.

Dipping the fermented plaque sample(s) into the Plaque Indicator Neutralizing Solution (dispensing dish section B) will cause the plaque sample to change back to a green colour. This demonstration can be used to educate the patient regarding the protective actions of saliva (which contains bicarbonate and other buffer systems).

After further motivational discussion, the clinician can design a homecare oral hygiene program with the patient which can help them remove plaque by proper brushing and interdental cleaning techniques.

REMOVAL OF PLAQUE DISCLOSING GEL

Instruct the patient in proper tooth brushing and thereby remove any remaining plaque and disclosing gel.

- NOTE:
- Take care not to confuse Plaque Indicator Solution in dispensing dish section A with Plaque Disclosing Gel placed in section C.
 - Clean the tip of each bottle or tube by wiping with a sponge or cotton pellet to remove any excess gel or solution. Close the cap immediately.
 - After patient testing is complete, collect all used items and dispose of carefully.

PACKAGE

Plaque Indicator Solution 3 mL (A)
Plaque Indicator Neutralizing Solution 3 mL (B)
Plaque Disclosing Gel 5 g tube (C)
Disposable dispensing dish 20 pieces
Disposable plaque collection instrument 40 pieces
Directions for use

STORAGE

Store at normal room temperature away from high temperature or direct sunlight.
Shelf life: 3 years from date of manufacture

CAUTION

- This product may cause sensitivity for some people. If any such reactions are experienced, discontinue the use of product and refer to them to a physician.
- Avoid contact of Plaque Indicator Solution, Plaque Disclosing Gel or Plaque Indicator Neutralizing Solution with eyes. In case of contact, immediately flush with water and seek medical attention.
- In case of contact with skin, immediately remove with a sponge or cotton pellet and wash with water.
- Avoid contact of any of the components with clothing or equipment. In case of contact, immediately wash with water.
- Do not use or mix with other products.
- Do not reuse the disposable dispensing dish or plaque collection instruments.
- The disposable plaque collection instrument has sharp tip. Take special care when using it in mouth.
- Plaque Indicator Solution should be protected from light. Dispense immediately before use.
- Do not use Plaque Indicator Solution or Plaque Indicator Neutralizing Solution in the mouth.
- Do not dispense Plaque Indicator Solution (A) into the same well of the dispensing dish as Plaque Indicator Neutralizing Solution (B).

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