




# MASK MOUTH

It is an emerging dental concern due to prolonged use of Mask

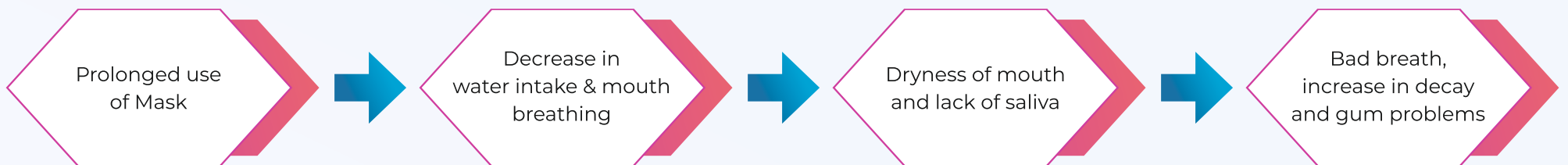

## Recommendations to prevent Mask Mouth?


- Use clean mask
- Drink plenty of water and keep yourself hydrated
- Improve your oral hygiene routine
- Use Tooth Mousse twice a day
- Take fresh air breaks
- Watch your diet

### What is Tooth Mousse\*?


A topical dental crème which stimulates salivation, prevents dental decay and strengthen tooth structures.



### How to use Tooth Mousse\*?

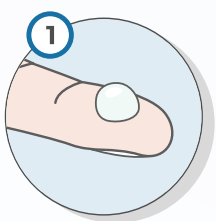


After brushing use finger or cotton to apply Tooth Mousse.

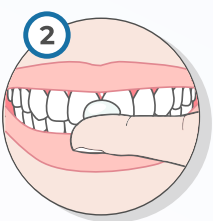

Use crème to coat all upper and lower teeth.



Leave for 3 mins and spread the crème all over the mouth with tongue.

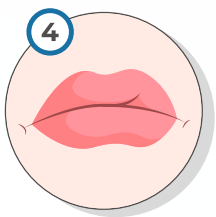

Hold in the mouth for 1-2 mins for effective result.



Spit out & no need to rinse.



Do not eat or drink for 30 mins after applying Tooth Mousse for better result.

\*ASK YOUR DENTIST OR ORDER ONLINE

### How your dentist can help you?

- ✓ Get yourself checked for any tooth decay and gum diseases.
- ✓ Get preventive Fluoride varnishes & Specialised dental cremes.
- ✓ Get your damaged tooth filled with fluoride releasing restorations.
- ✓ Get your dental cleanings done.



#### Tooth Mousse

Topical creme for decay prevention and strengthen tooth structures.



#### MI Varnish

Fluoride varnish for decay prevention



#### Dry Mouth Gel

Helps to relieve dry mouth



#### Fuji VII

Pit & Fissure decay sealant



#### Glass Ionomers & Glass Hybrids

Fluoride releasing restorative cements

Consult your dentist for prevention & treatment

PHOTOGRAPH

If you think you have mask mouth, don't ignore it.

**GC**



Since 1921  
100 years of Quality in Dental

www.gcindiadental.com